





Accessible | Affordable | Functional

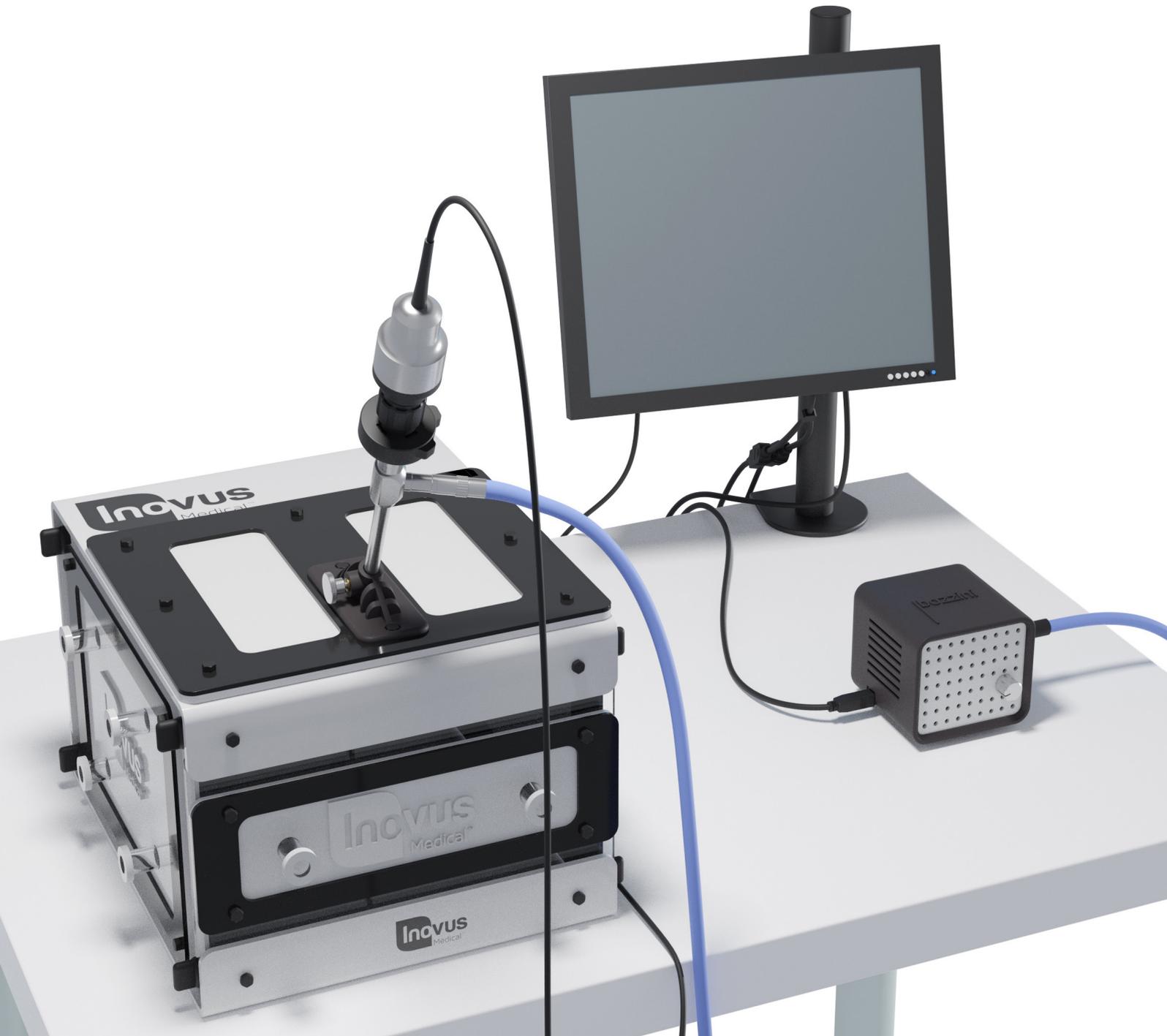

Bozzini Laparoscopic Simulator Setup Manual

Product number: 00000960

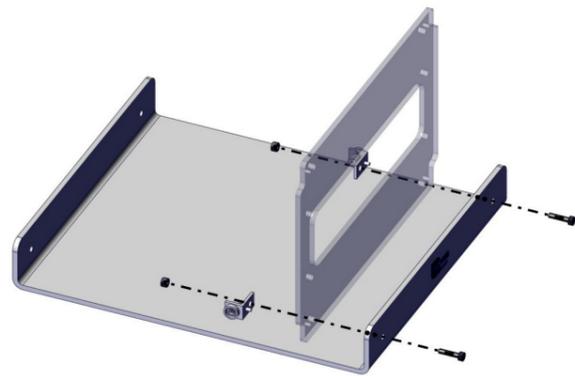


1 | Construct the front panel section of the simulator.

You will need parts: 00001279, 00000939 and 2 x 00000943.

Using 2 x 00000640 bolts and 2 x 00001159 nuts, assemble as shown in the image.




4 | Your simulator should now look like this, with both upright panels securely in position.

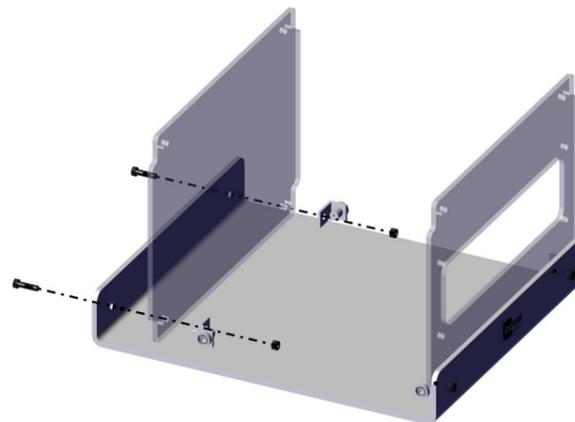

2 | Repeat Step 1 for the rear panel 00000940.

You will need to use 2 x 00000943, 2 x 00000640 bolts and 2 x 00001159 nuts as for Step 1.

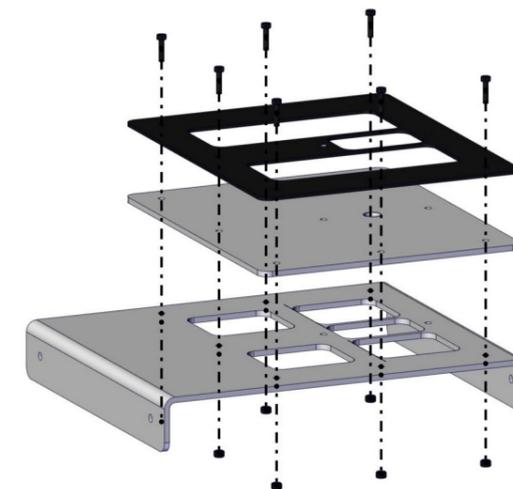



5 | You may now start the top panel sub-assembly.

Taking 00000926, 00000933 and 00001278, secure as shown using 7 x 00000640 bolts and 7 x 00001159 nuts.

3 | You will now need the following parts:
 - 00000935
 - 00000927

Using 4 x 00000640 bolts and 4 x 00001159 nuts, secure 00000640 and 00000927 to the 00000939 front panel.


Please note that the black bracket must be placed on the outside of the white skin as shown in the image.

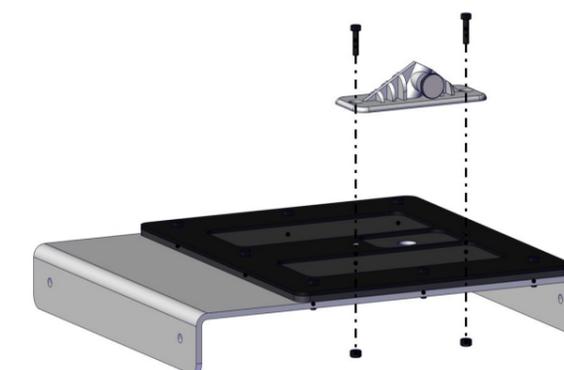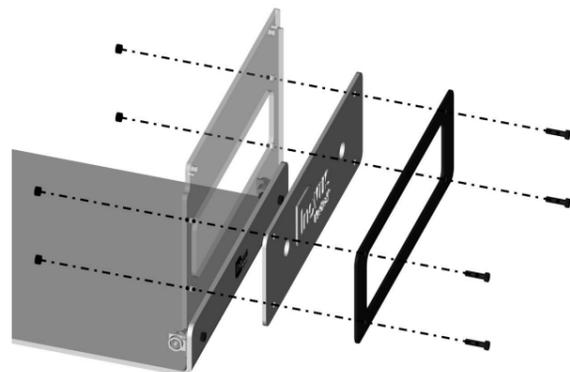


6 | Isolate the scope bracket (00001280) Using 2 x 00000640 bolts and 2 x 00001159 nuts, secure the bracket as shown in the image.

You have now completed the top panel sub-assembly.


Please note you may operate without the bracket when a camera operator is present.

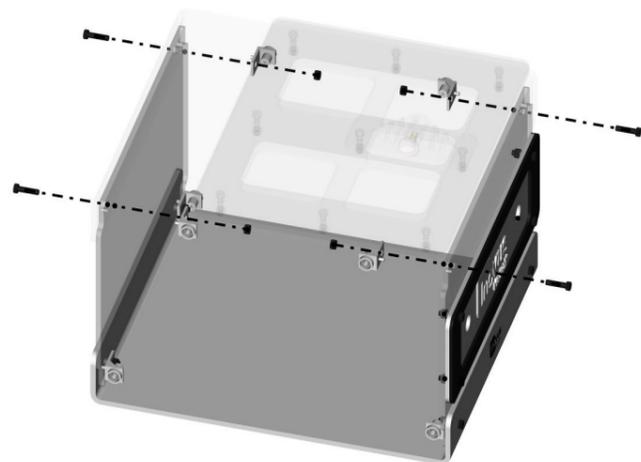


7 | As in Steps 1 and 2, secure the top panel sub-assembly to the upright panels using: 4 x 00000943, 4 x 00000640 bolts and 4 x 00001159 nuts.

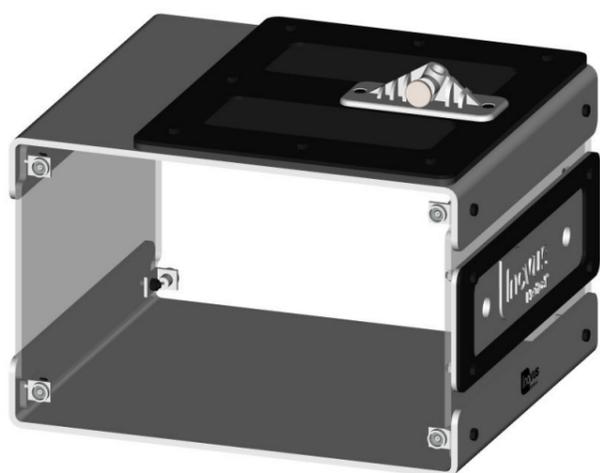

8 | Your simulator should now look exactly like the image shown.

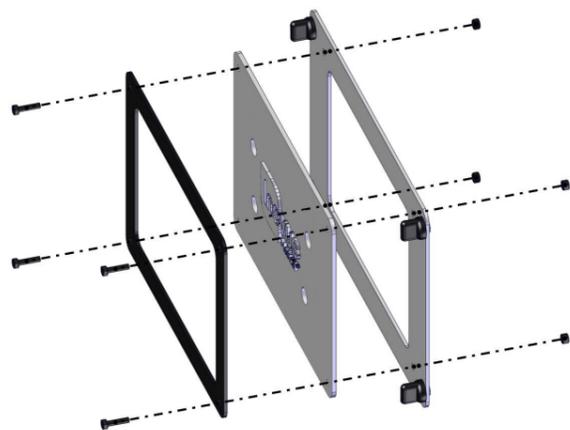


9 | You may now start the side panel sub-assemblies. Taking 00000946, 00000640 and 00000929, secure as shown using 4 x M529 bolts and 4 x WN29 nuts.

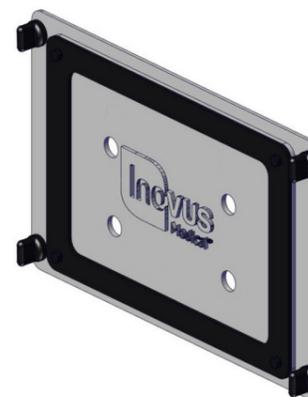

10 | Your side panel should look like this. Repeat the process one more time for the opposite side.

Slot the side panels into position, allowing the magnets to guide the panel into place.

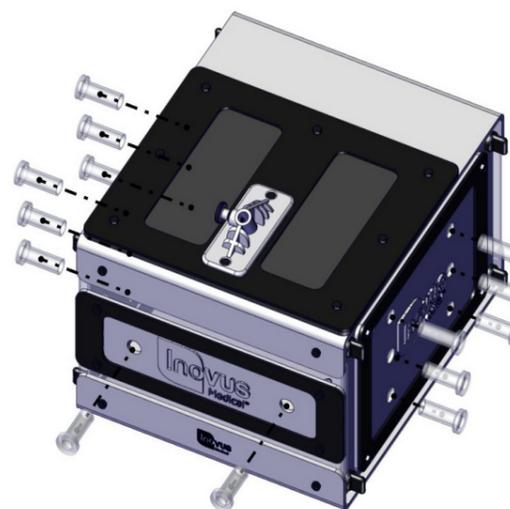


11 | Insert the 10 dummy trocars as directed in the image.



12 | You are now ready to assemble the monitor bracket. Begin by removing the C-clamp from the package.


If your simulator was purchased in a hard shell carry case, the monitor bracket will be assembled already and you may skip to Step 20.

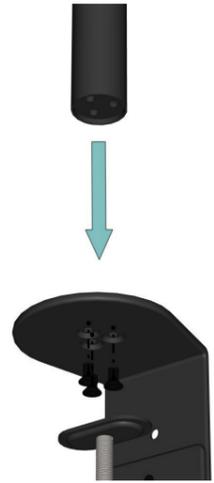


13 | Using the three black bolts provided, attach the black post to the C-clamp.



16 | At the desired height, use the largest Allen key to lock into position by turning the bolt in a clockwise direction.

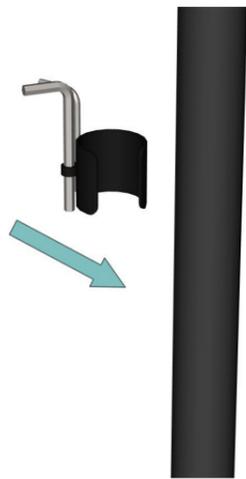

14 | Just so that we know where they are for later, identify the two Allen keys along with the push fit clamp provided. Snap this to the post and move to the next step.

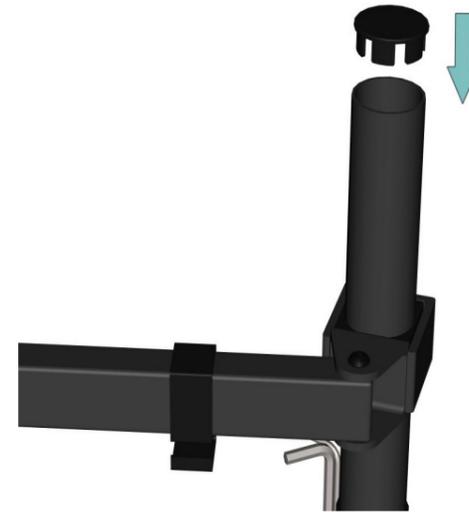


17 | Push the end cap into position.




15 | The swivel arm should be pre-assembled. Slide this onto the black post, then move to the next step.



18 | With the bracket fully assembled, anchor it to your chosen work surface by turning the handle until it is tight.

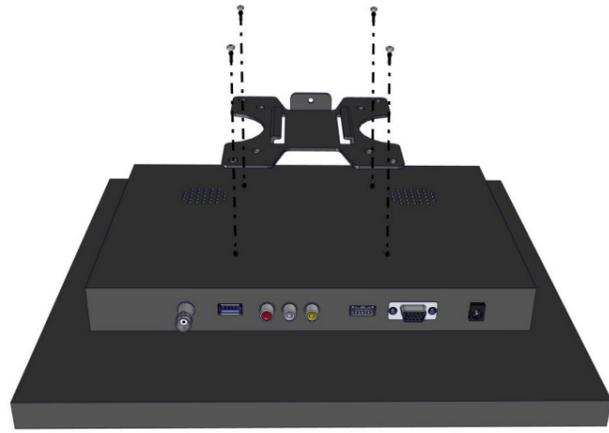

19 | Take the screen bracket and screw this to the back of the monitor using the bolts provided.

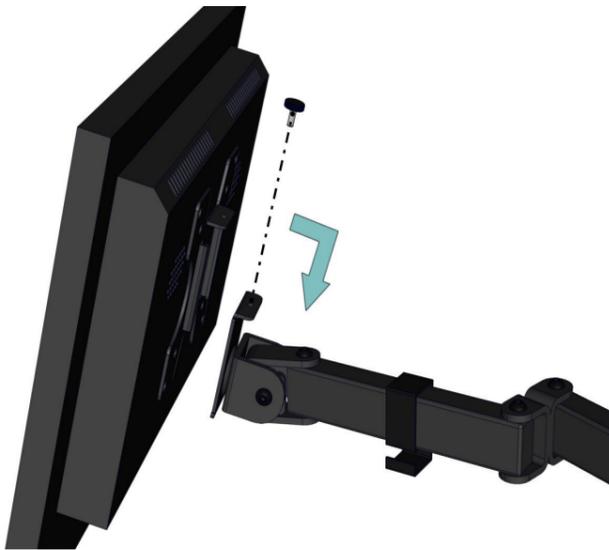


20 | Slide the monitor into position and screw the bolt into place.



21 | All wiring can be completed using the numbered stickers attached to the ends of each adapter. For additional support, use this visual guide.

Start by plugging in the 12V power adapter.

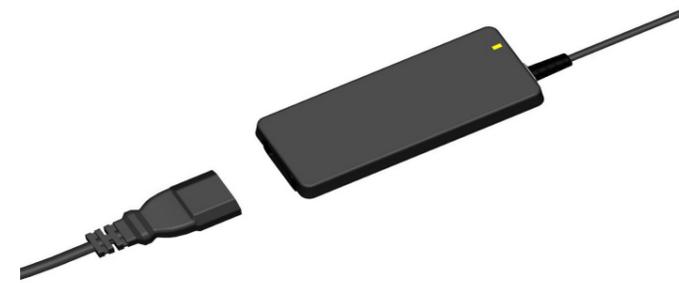


22 | Plug the socket into the wall and then connect the IEC cable to the power adapter. When switched on, a green light may be shown.



23 | Connect the USB splitter to the rear of the monitor as shown.



24 | Connect the USB to 5V adapter to one of the USB ports. Connect the female end of the BNC cable to the male end of the 5V adapter.

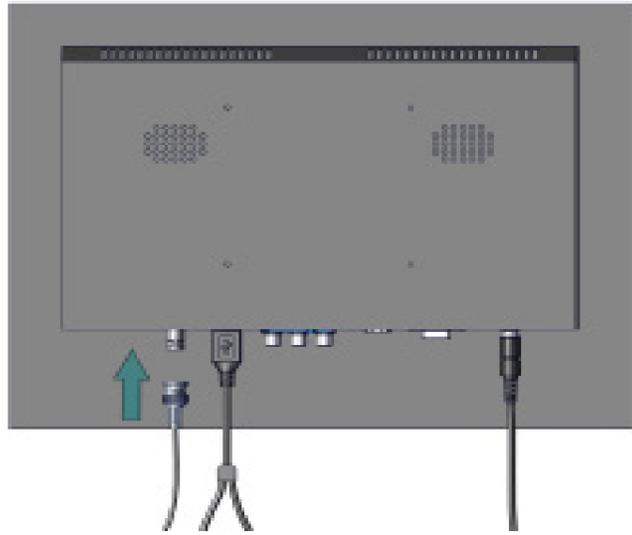


25 | Connect the female end of the BNC extender to the screen.




26 | Locate the endoscopic camera in the package. Connect the opposing ends of the BNC cable to the camera connectors.




27 | Connect the USB A/B cable to the remaining USB port on the USB splitter.

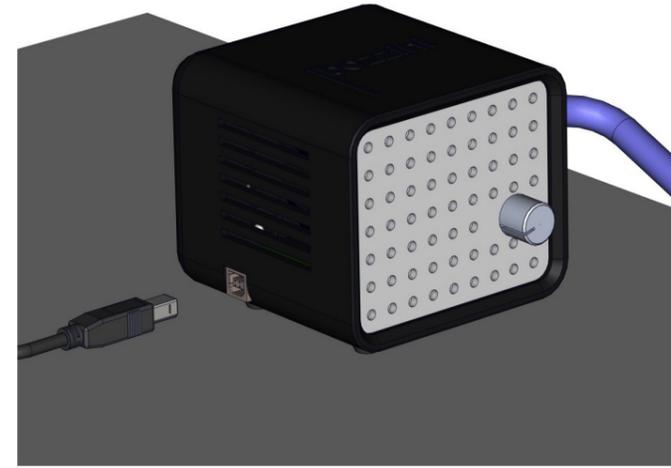


28 | Now connect this wire to the bozzini light source.

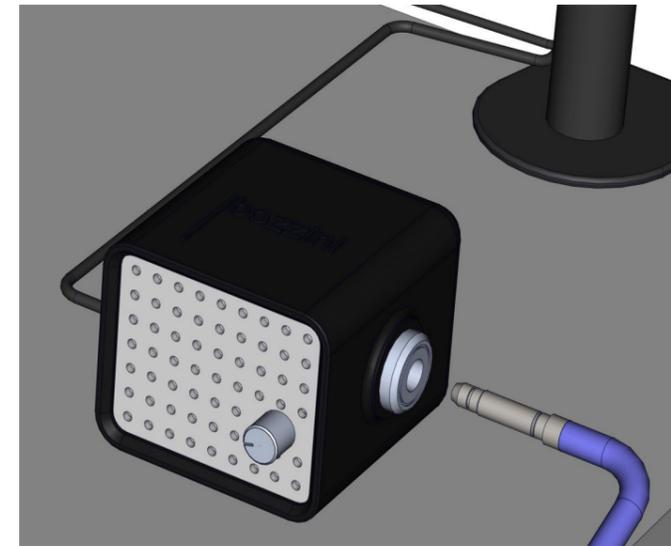

29 | Plug in the male end of the fibreoptic cable into the light source.

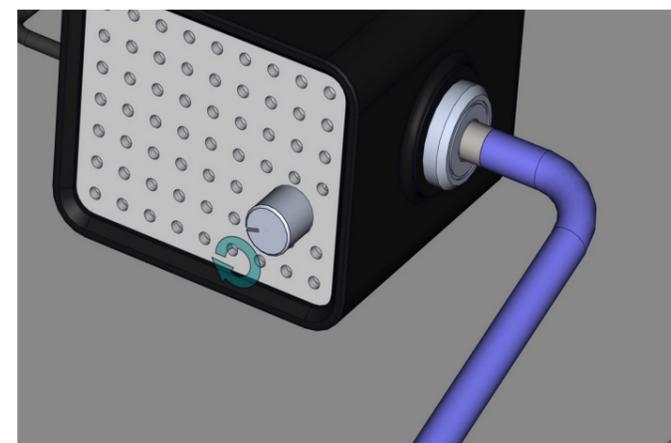


30 | When the screen is powered on, you should be able to turn the knob on the front of the light source to activate the light. Test the light at this stage and switch off.

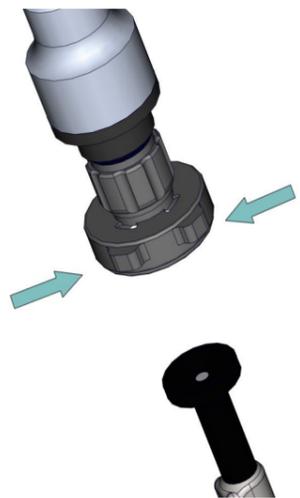

31 | Connect the camera coupler to the laparoscope by pinching the two clasps.


It should lock into place firmly.

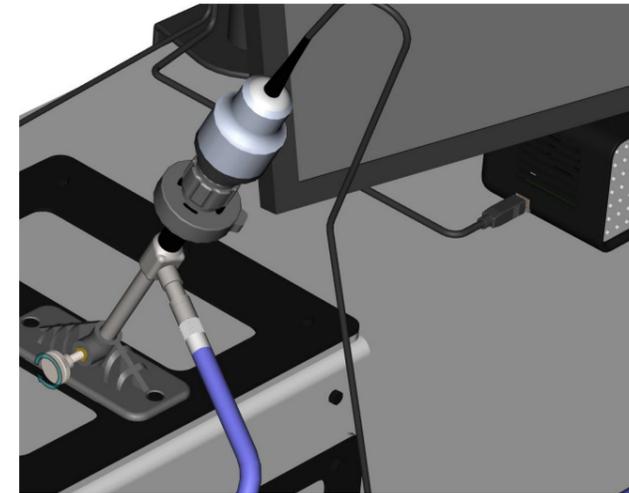

34 | The scope system should look something like this.



32 | Slide the laparoscope into the bracket and lock into position by tightening the thumb screw.

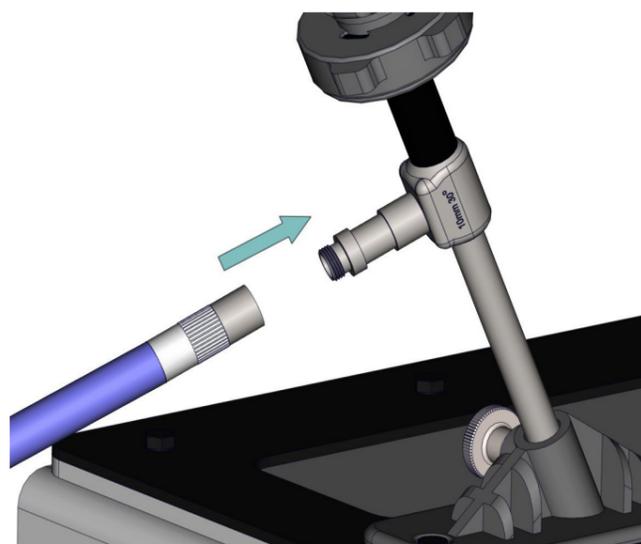

33 | Attach the other end of the fiberoptic cable to the connector on the scope by screwing it into place. At this stage, you are ready to turn the light source on.



35 | In most cases, your monitor will have been pre-programmed to display the image. In the event that you do not have an image on screen, push the left most button on the screen and select BNC1.

Once the image appears you are now ready to operate.



Surgical Training Remastered

Global Headquarters
St Helens, United Kingdom
E: info@inovus.org
T: +44 (0)1744 752 952
www.inovus.org

USA Headquarters
Tampa, Florida USA
E: info@inovus.org
T: 512 481 7691
www.inovus.org

   @inovusmedical

All products contained in this catalogue are Copyright Inovus Ltd 2024
Full price list available on request: info@inovus.org