




Accessible | Affordable | Functional

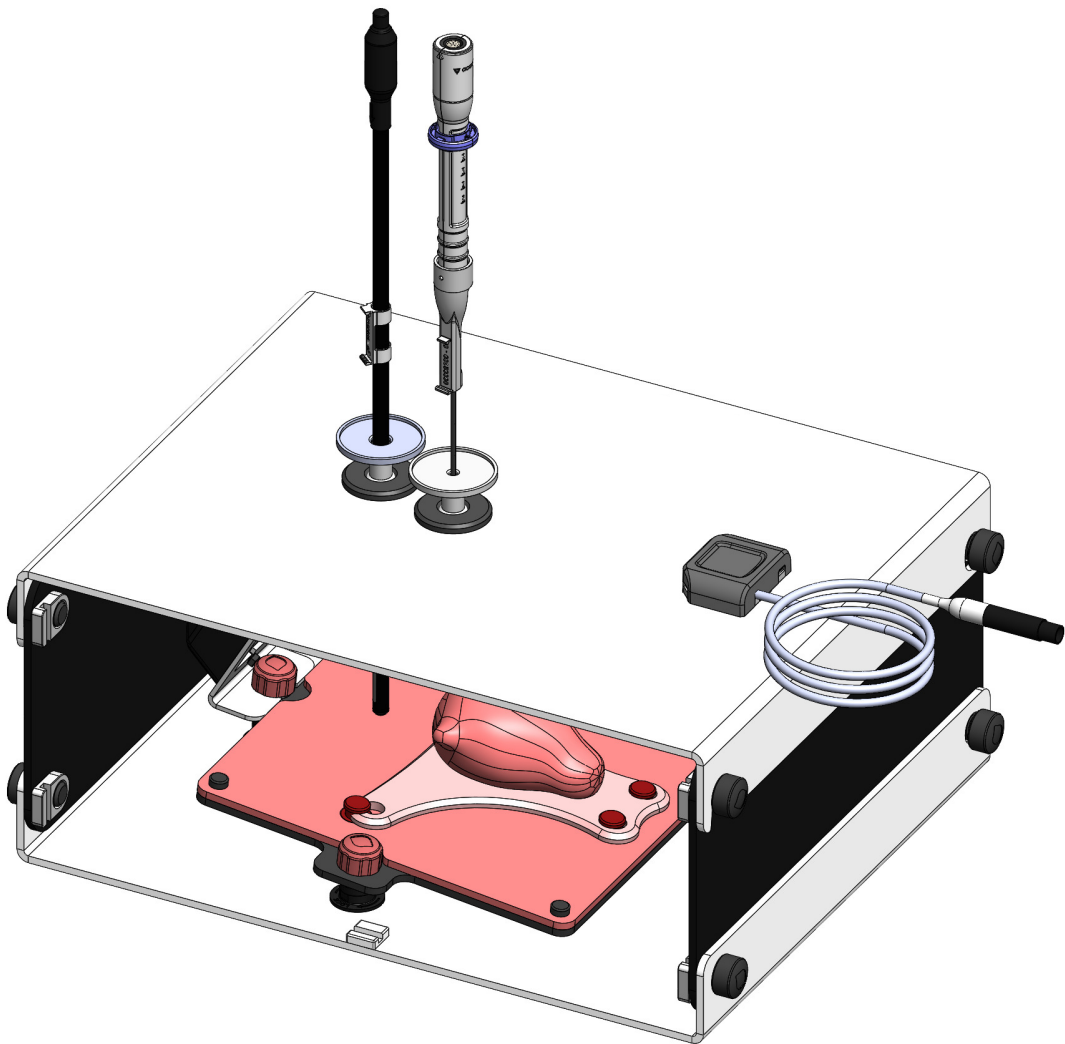


ACCESSASim

Setup Manual

Product number: 00001600


1 | Position the Base

- Place the base component on a stable, flat surface as shown in the diagram.

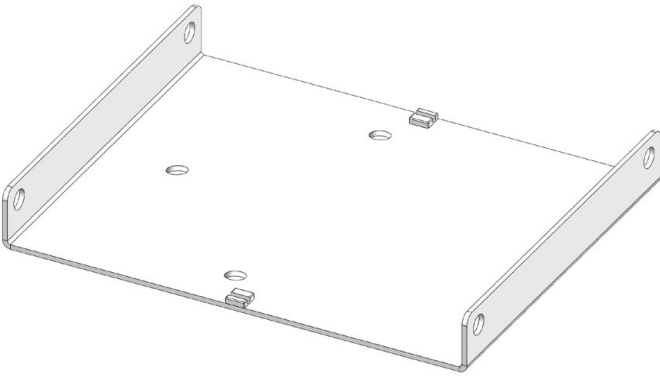


2 | Install Mounting Posts

- Remove the red caps from each mounting post.
- Insert the mounting posts into the base plate with the letter A on the top of each pillar facing forwards.
- Rotate and lock them securely into place.

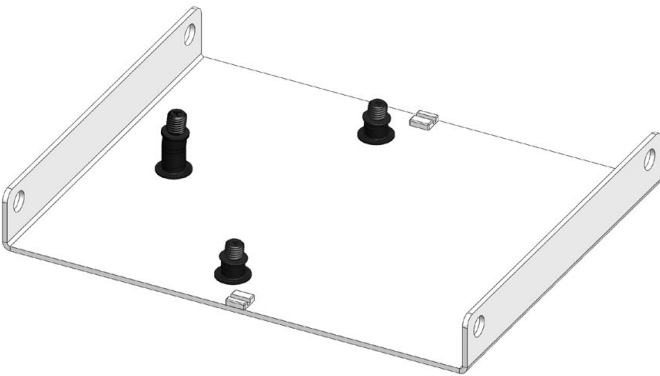


3 | Attach the Model Mounting Jig

- Place the model mounting jig over the mounting posts.

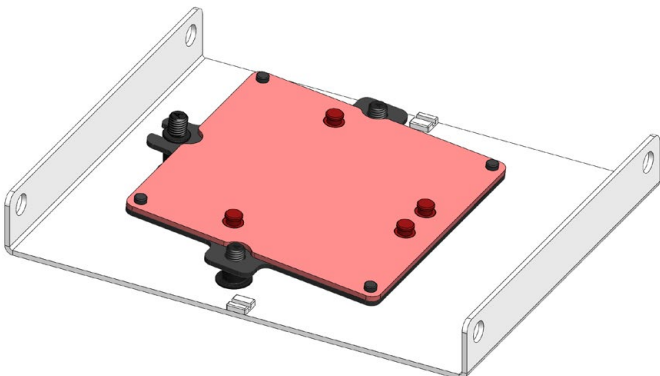


4 | Reattach red Caps to Side Posts

- Screw the red caps onto the two side mounting posts.

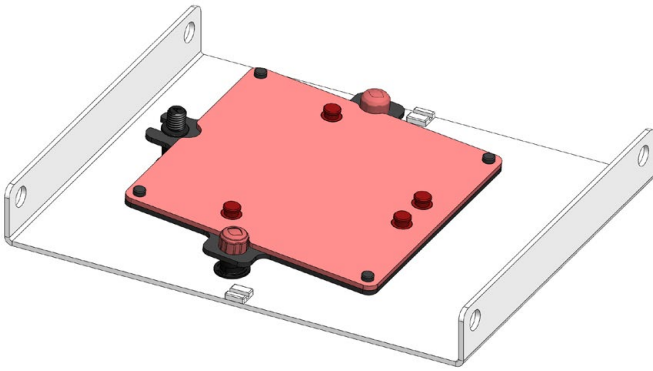


5 | Install the Camera Bracket


- Attach the camera bracket to the top mounting post.
- Ensure it is angled toward the red plate.

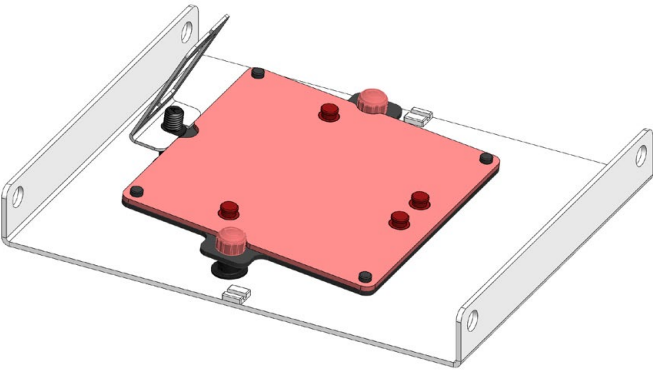

6 | Secure the Camera Bracket


- Screw the red cap onto the top mounting post, over the bracket.

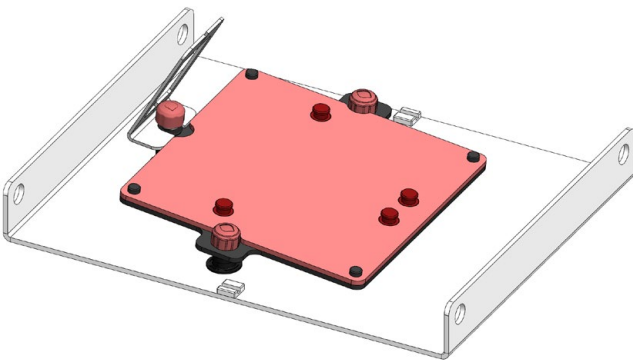

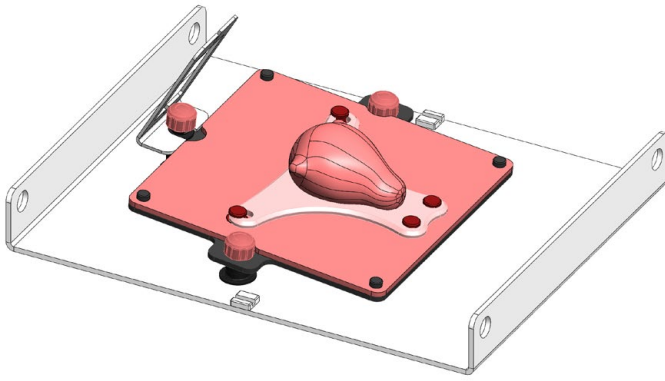


7 | Mount the Uterus Model

- Place the uterus model onto the model mounting jig.

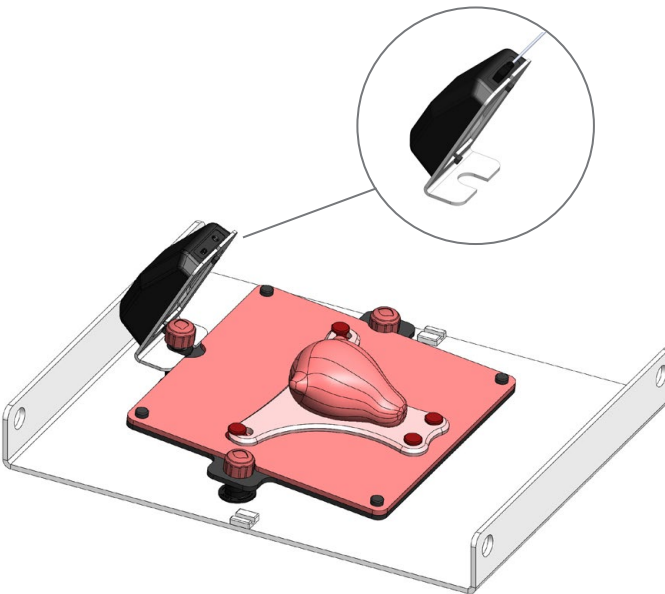


8 | Install the Camera

- Slide the camera into the bracket, aligning screws with bracket openings.
- Plug in the camera using a micro-USB to USB-A cable.
- Ensure the cable is facing upwards.

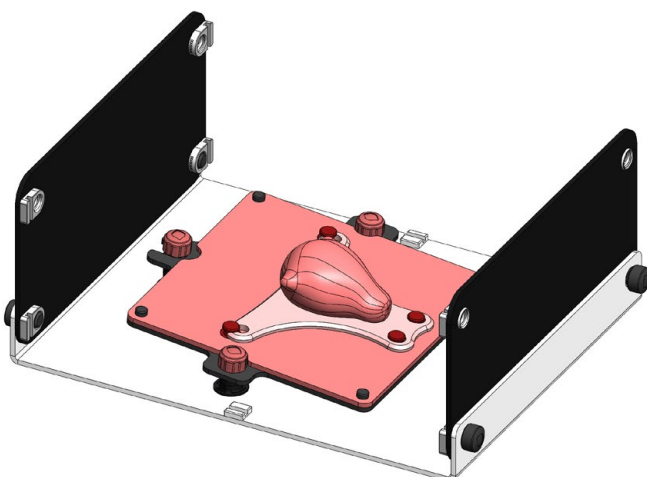


9 | Attach the Front and Back Panels

- Place the front black panel inside the white lip of the base plate. The Hologic logo should face outward.
- Insert and fasten the black screws from the outside.
- Repeat for the back panel.

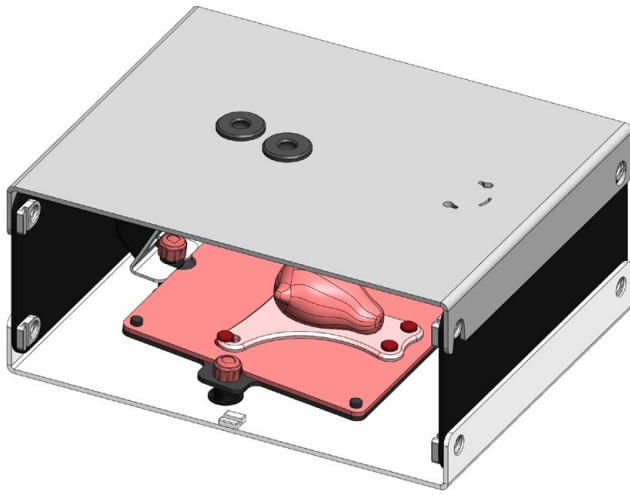


10 | Attaching the top panel

- Position the top panel so that the trocars align with the fundus of the uterus model.
- Ensure the top panel sits on the outside of the front and back panels.

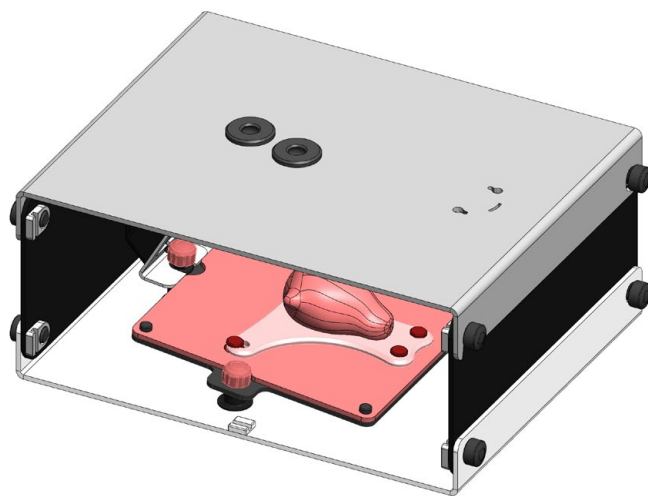


11 | Secure the Top Panel

- Use the remaining four screws to secure the top panel to the front and back panels.

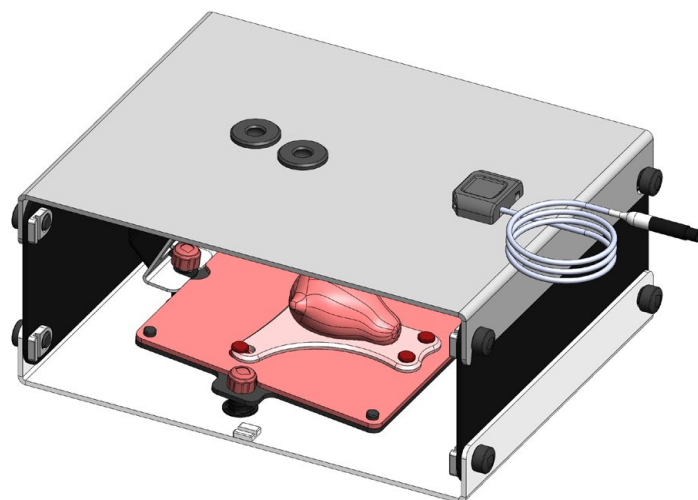


12 | Install the USB Hub

- Orientate the USB hub with the standard USB port facing the trocar openings.
- Insert the fixed screws on the underside of the USB hub into the top panel, and rotate clockwise to lock.

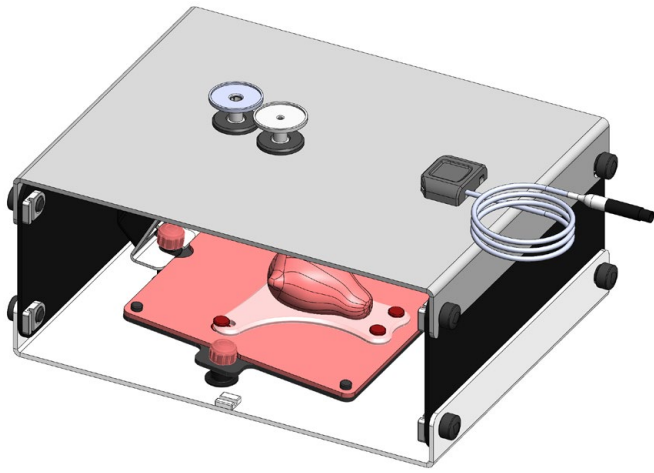


13 | Insert the trocars

- Insert the smaller-opening trocar for the handpiece.
- Insert the larger-opening trocar for the transducer.
- Make sure the round shaft sits inside the black rubber grommet (do not push fully in).

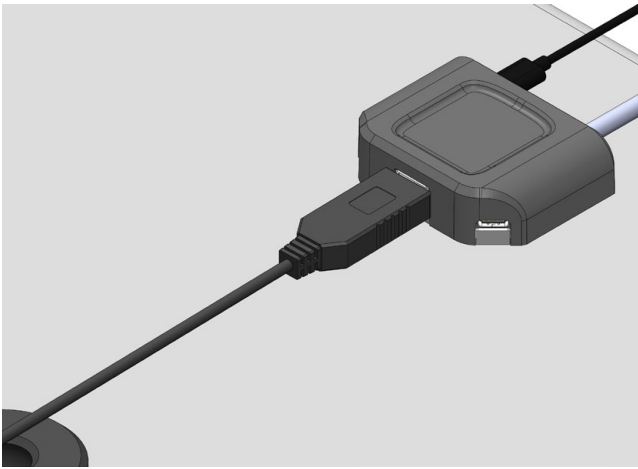


14 | Connect the Camera to the USB Hub

- Plug the USB end of the camera cable into the USB hub.

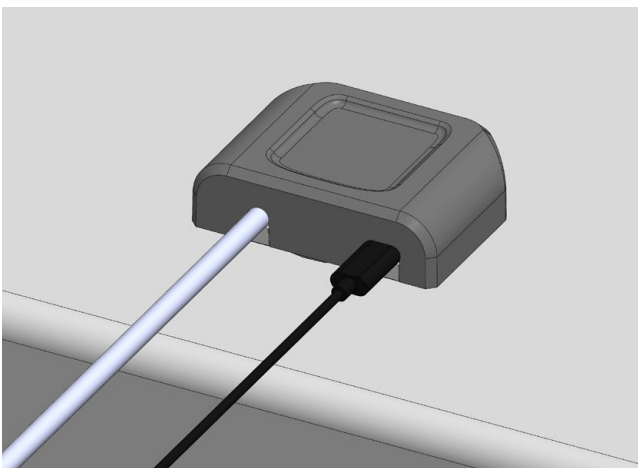


15 | Connect USB-C to the Hub

- Use the long USB-C to USB-C cable and connect it to the back of the USB hub.

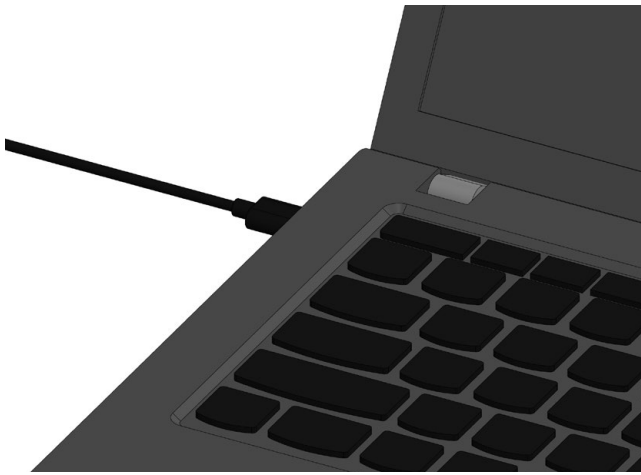

16 | Connect to Laptop

- Plug the other end of the USB-C cable into your computer.

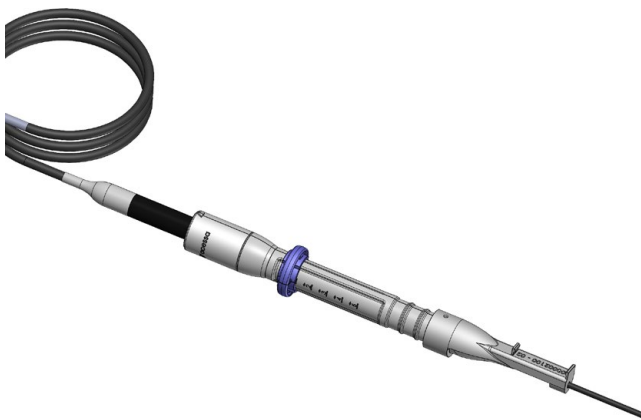

17 | Connect the Acesa Handpiece

- Attach the Acesa handpiece to the appropriate cable from the USB hub.

Launching the Totum Application

- Ensure your laptop is connected to Wi-Fi.
- Open the Totum application on your desktop.
- Log in using your assigned credentials.



Surgical Training Remastered

Global Headquarters
St Helens, United Kingdom
E: info@inovus.org
T: +44 (0)1744 752 952
www.inovus.org

USA Headquarters
Tampa, Florida USA
E: info@inovus.org
T: 512 481 7691
www.inovus.org

