



Mac Setup Guide:

Step 1: Click the search icon and type 'quick time player'

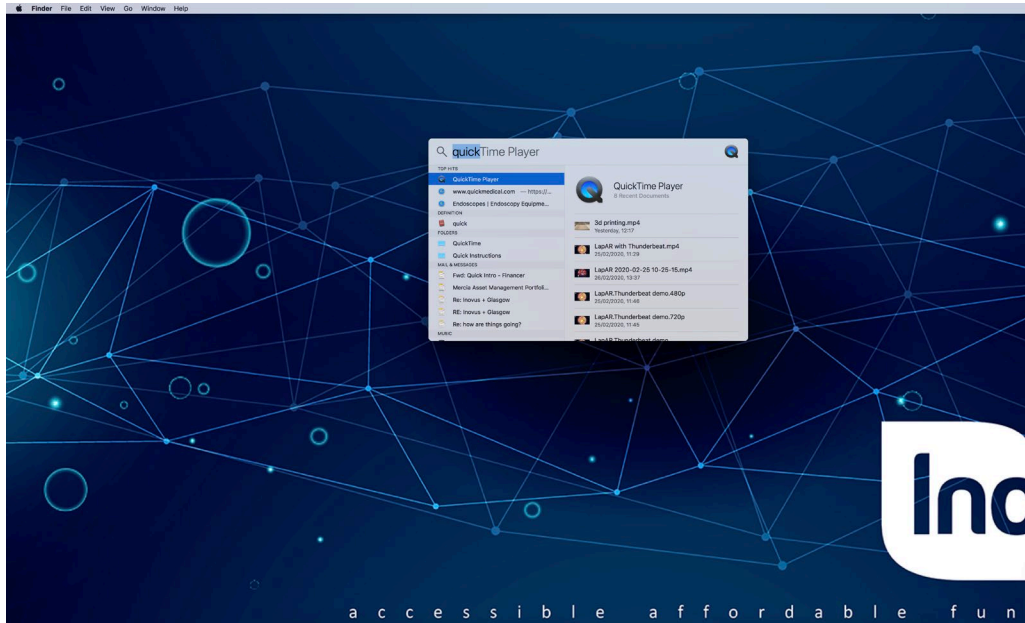

Step 2: Click the QuickTime icon highlighted by the red circle.

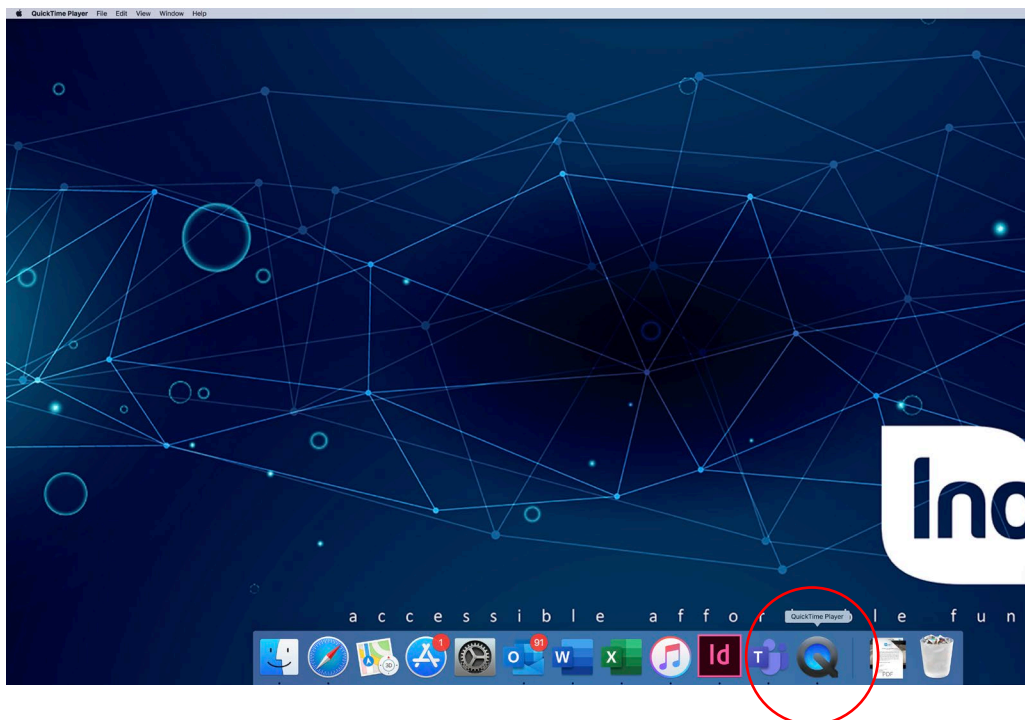

a c c e s s i b l e a f f o r d a b l e f u n c t i o n a l

E info@inovus.org


T +44 (0)1744 752 952

W www.inovus.org


All products are Copyright Inovus Ltd 2020

 @inovusmedical

Step 3: Click file > New movie recording

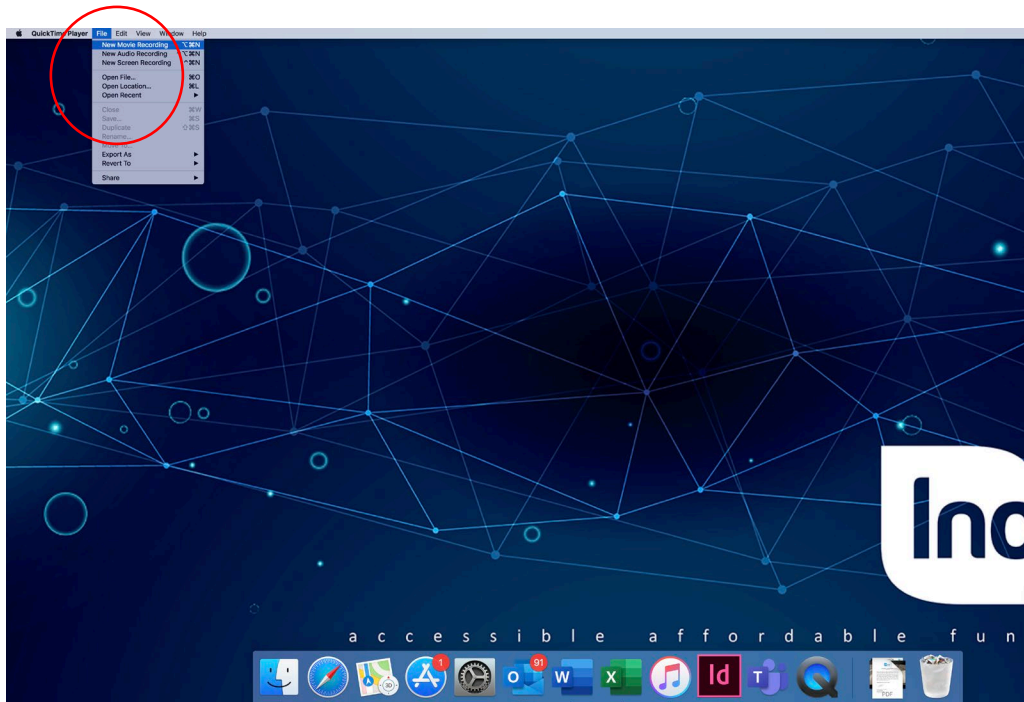

Step 4: If your system doesn't automatically recognise the camera you will need to change the default camera in step 5.

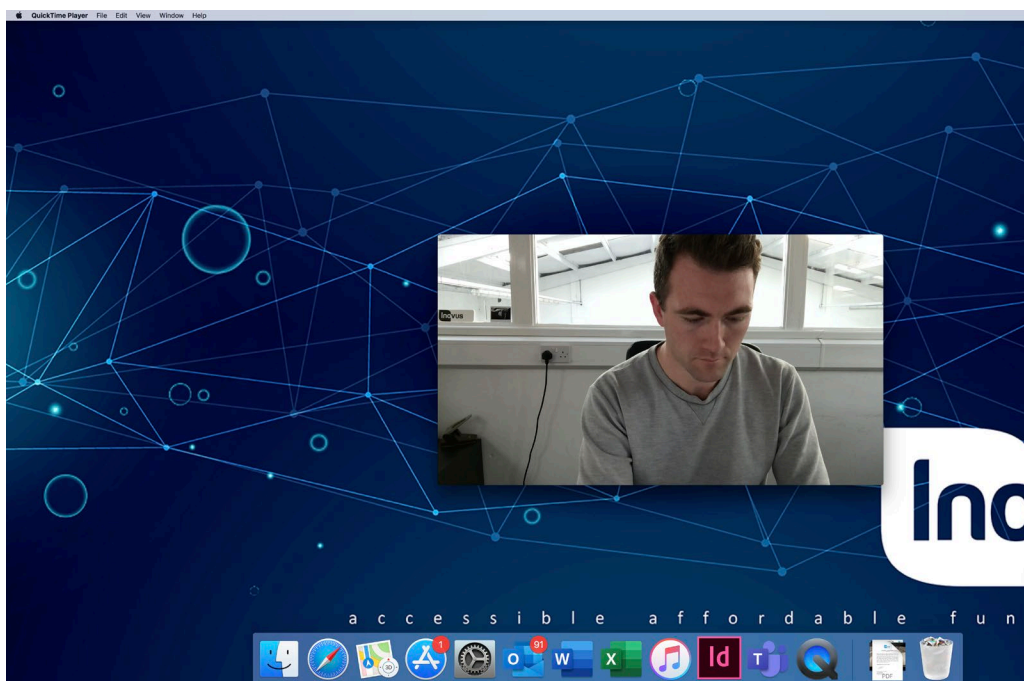

a c c e s s i b l e a f f o r d a b l e f u n c t i o n a l

E info@inovus.org


T +44 (0)1744 752 952

W www.inovus.org


All products are Copyright Inovus Ltd 2020

 @inovusmedical

Step 5: Click the down arrow to the right of the record button and select USB 2.0 Camera

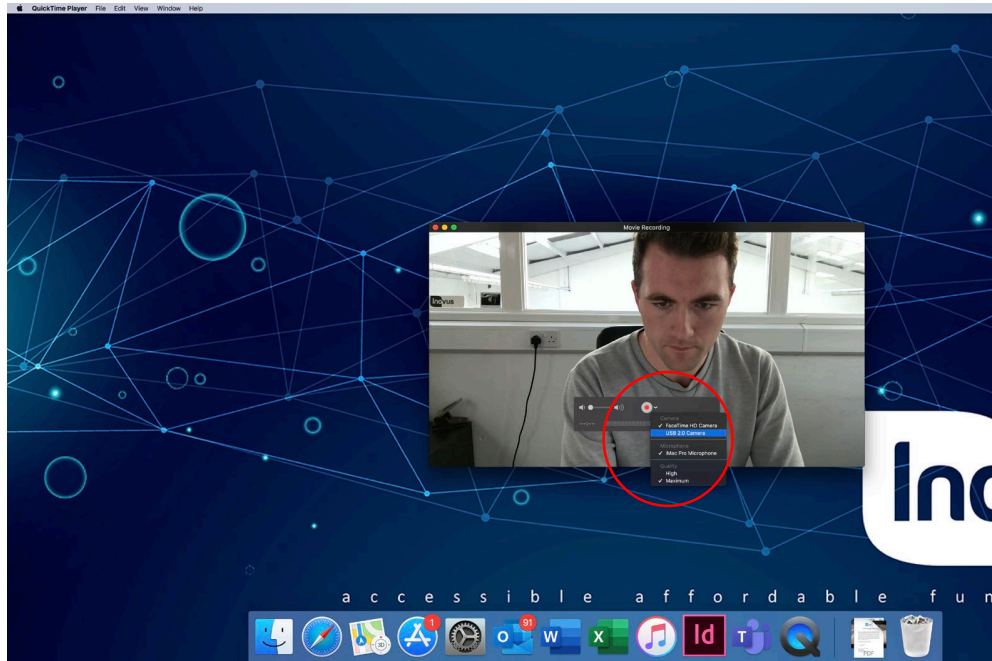

Step 6: Once the camera opens maximise the screen and begin to operate. You can also change the quality settings as shown below.

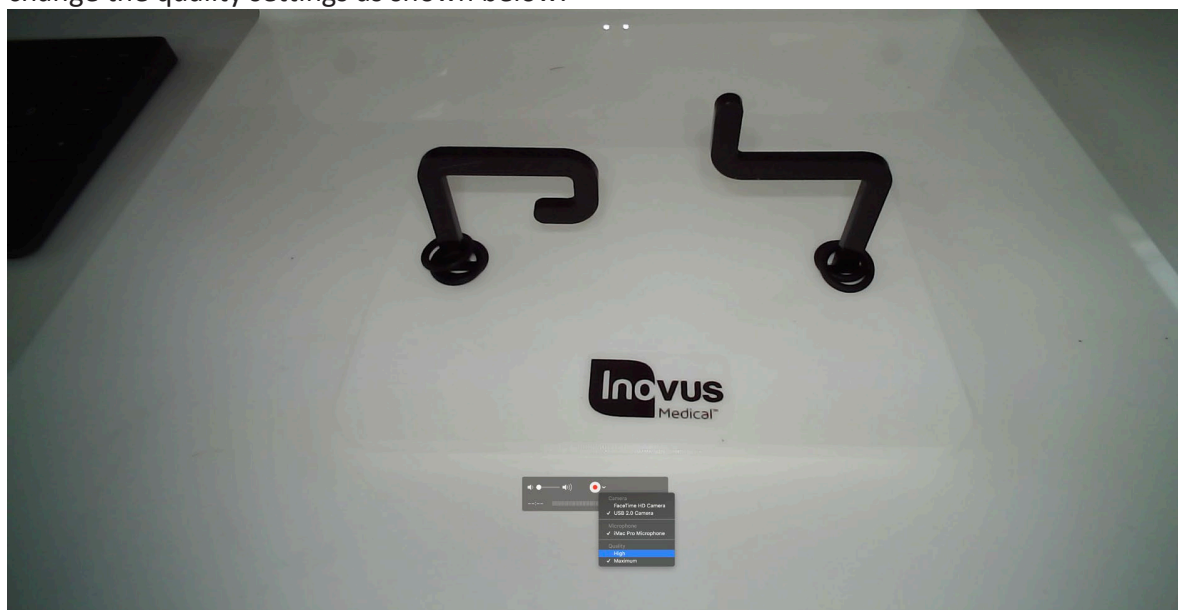

a c c e s s i b l e a f f o r d a b l e f u n c t i o n a l

E info@inovus.org

T +44 (0)1744 752 952

W www.inovus.org

All products are Copyright Inovus Ltd 2020

 @inovusmedical