

sellick®



Inovus
Medical®

a c c e s s i b l e a f f o r d a b l e f u n c t i o n a l



- 1 | Open the box and remove the base unit, neck section and wires.



- 2 | Place the neck section into the recess in the base unit. Make sure the thyrod end of the neck section points towards the power input end of the device.




- 3 | Ensure the sellick® is placed on a desk or table top and then insert the USB power source.



- 4 | Once plugged into the power the display lights should show up. If numbers do not appear, press down once on the neck section to re-set the device.



- 5 | Press down on the neck section and pressure (Newtons) will read out in real time.

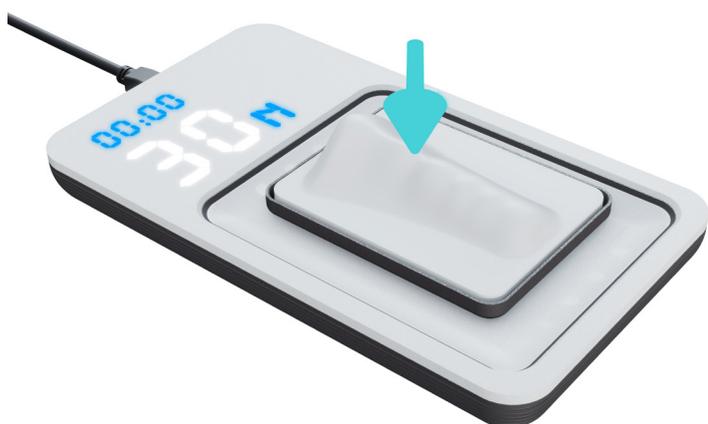

- 6 | The timer starts with pressure and stops on release. To re-set the timer push down and release the neck section once.



- 7 | Excess cricoid pressure over 35N will cause flashing red lights to appear.



- 8 | Lateral and rostral pressure is indicated by constant blue lights.

Designed and manufactured by



E: info@inovus.org

T: +44 (0) 1744 752 952

W: www.inovus.org



a c c e s s i b l e a f f o r d a b l e f u n c t i o n a l