




Accessible | Affordable | Functional




# Bozzini Hysteroscopy Simulator Setup Manual


Product number: 00001265

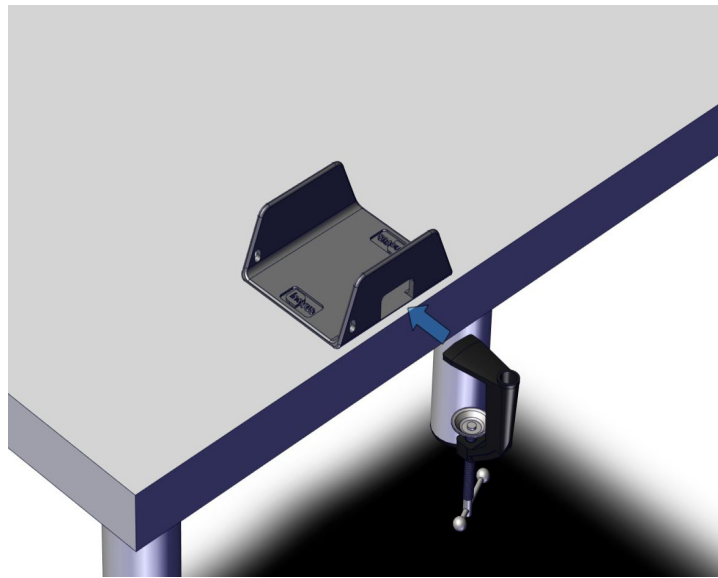


1 | Begin by placing the U-shaped bracket on a suitable flat surface. Then, place the top portion of the small clamp through the opening at the front of the bracket, as illustrated here.

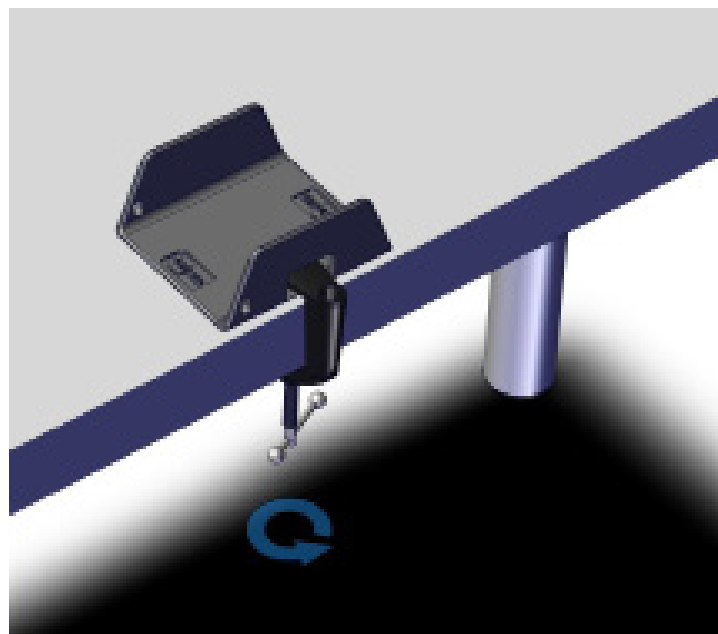


2 | Once in position, turn the handle until the clamp grips firmly to the bracket and flat surface.

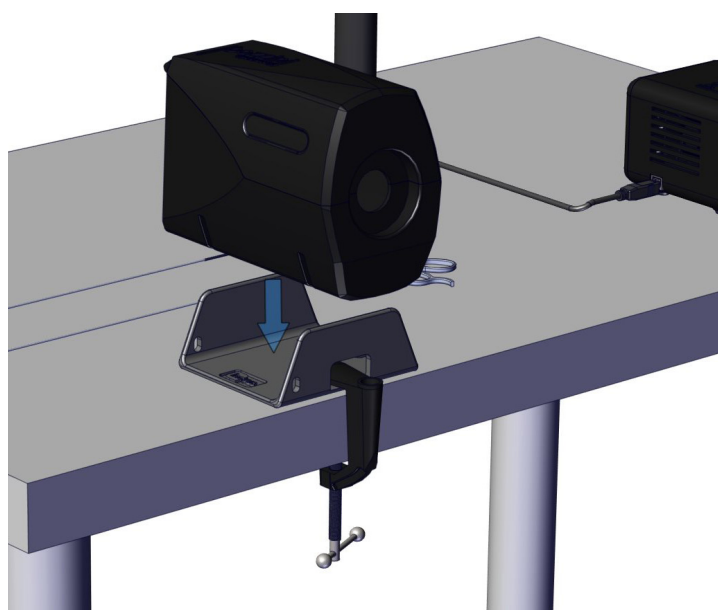


3 | Place the retaining case onto the U-shaped bracket as shown.

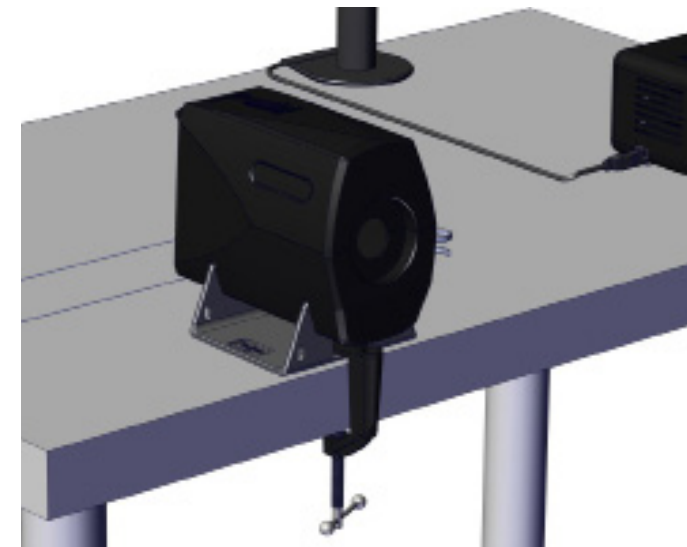

4 | The set-up should now look similar to this image.

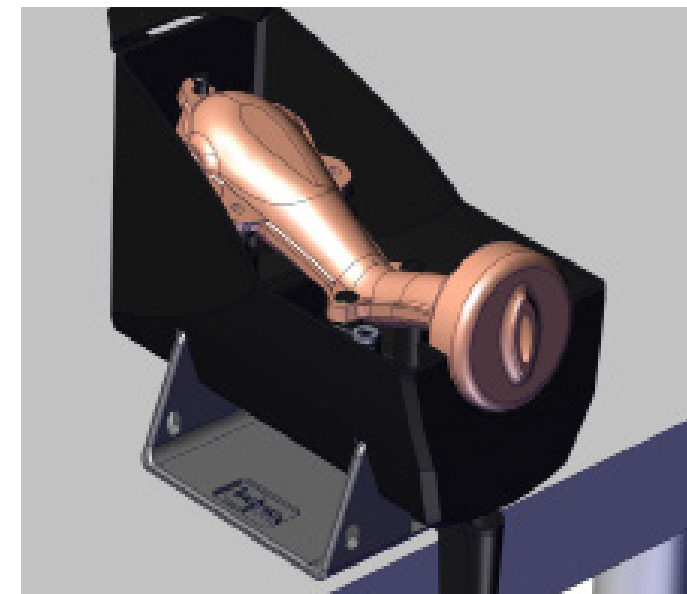

5 | You are now ready to place a uterus model inside the case. Hook the loops of the model onto the internal fixings so that it is held securely in place.

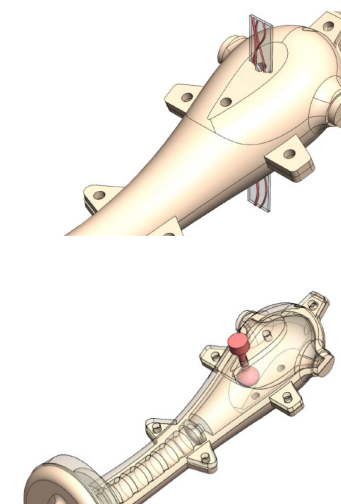

6 | At this stage you may introduce pathologies into the uterus, if your selected model allows you to do so.


Adhesions are inserted by placing the thread through one of the top holes and then pulling it through the corresponding hole below.

Polyps are inserted in the same way, however they do not need to be pulled all the way through the uterus.




7 | You are now ready to assemble the monitor bracket. Begin by removing the C-clamp from the package.


If your simulator was purchased in a hard shell carry case, the monitor bracket will be assembled already and you may skip to Step 15.

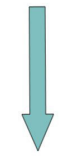

8 | Using the three black bolts provided, attach the black post to the C-clamp.

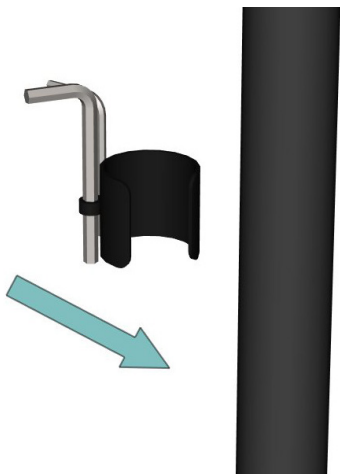


9 | Just so that we know where they are for later, identify the two Allen keys along with the push fit clamp provided. Snap this to the post and move to the next step.

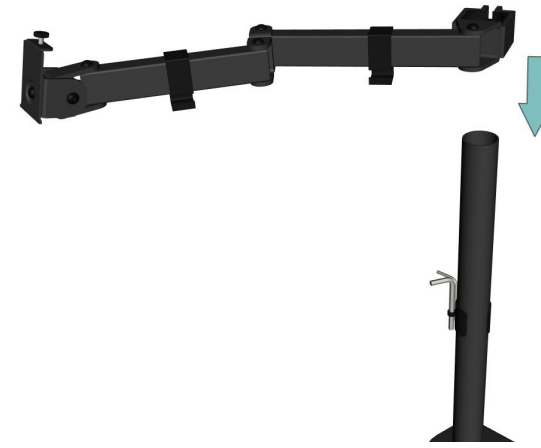


10 | The swivel arm should be pre-assembled. Slide this onto the black post, then move to the next step.




11 | At the desired height, use the largest Allen key to lock into position by turning the bolt in a clockwise direction. Once this step is complete, you may insert the end cap.

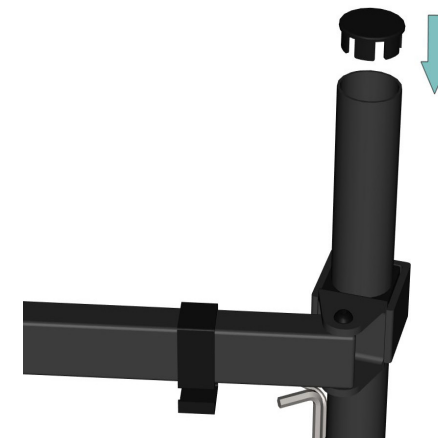


12 | Push the end cap into position.

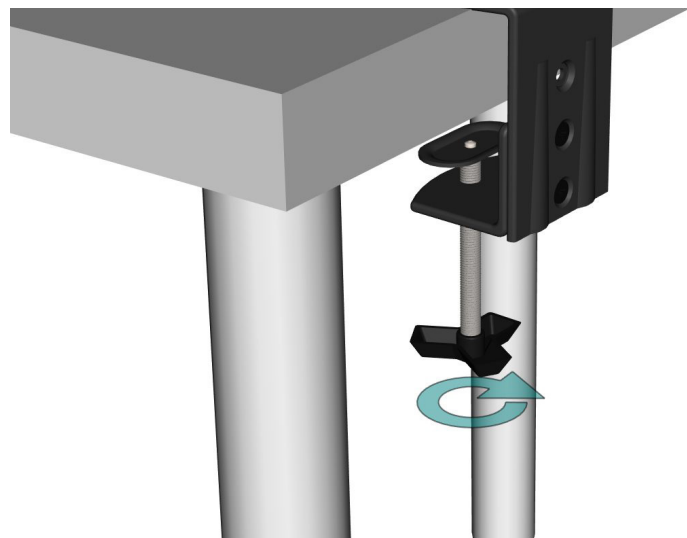

13 | With the bracket fully assembled, anchor it to your chosen work surface by turning the handle until it is tight.

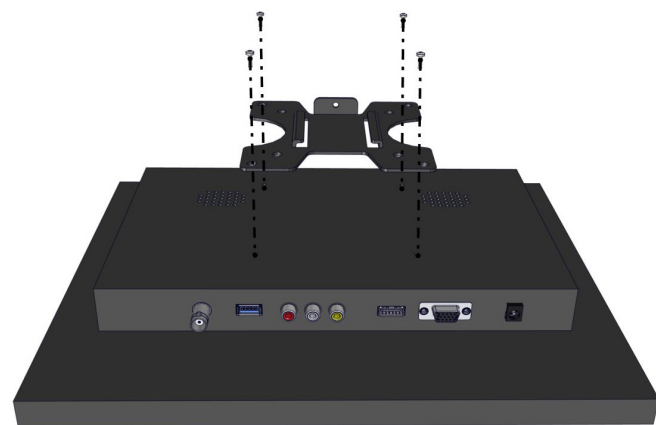

14 | Take the screen bracket and screw this to the back of the monitor using the bolts provided.

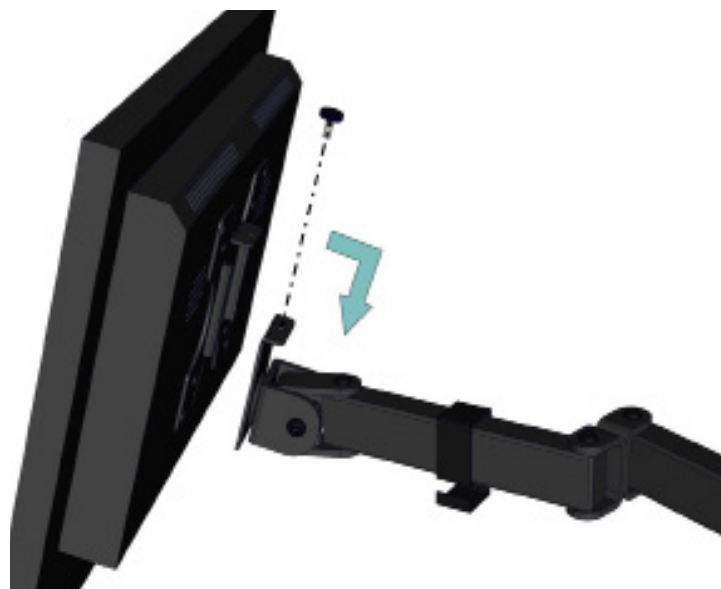

15 | Slide the monitor into position and screw the bolt into place.



16 | All wiring can be completed using the numbered stickers attached to the ends of each adapter. For additional support, use this visual guide.


Start by plugging in the 12V power adapter.



17 | Connect the USB splitter to the rear of the monitor as shown.



18 | Connect the USB to 5V adapter to one of the USB ports. Connect the female end of the BNC cable to the male end of the 5V adapter.

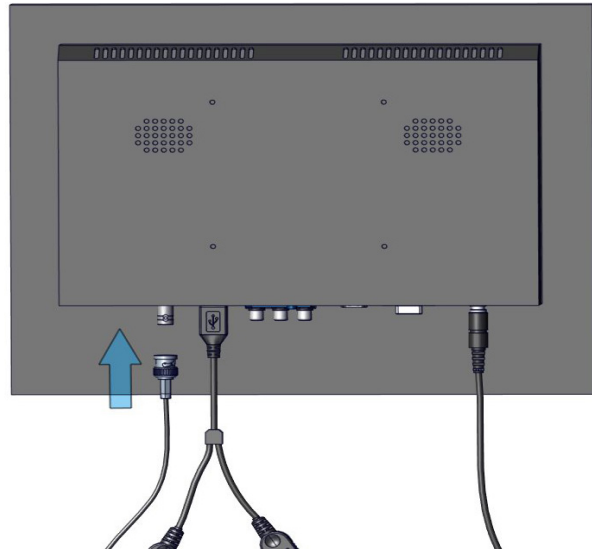

19 | Connect the female end of the BNC extender to the screen.




20 | Locate the endoscopic camera in the package. Connect the opposing ends of the BNC cable to the camera connectors.




21 | Connect the USBA/B cable to the remaining USB port on the USB splitter.



22 | Now connect this wire to the bozzini light source.

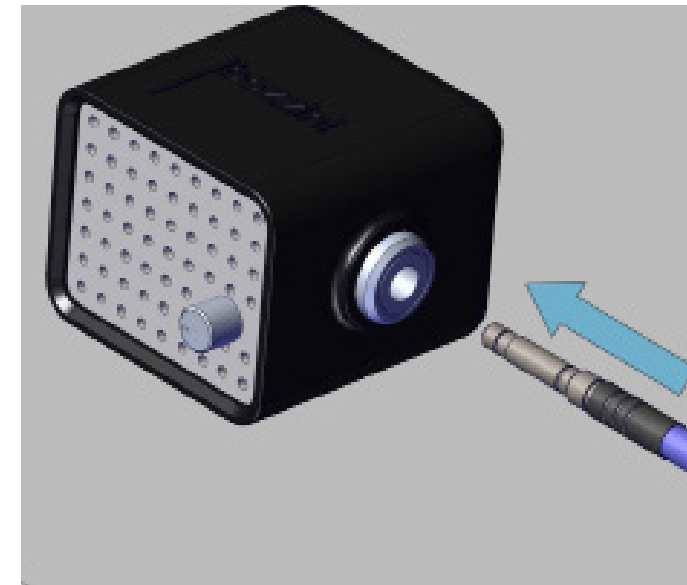


23 | Plug in the male end of the fibreoptic cable into the light source.

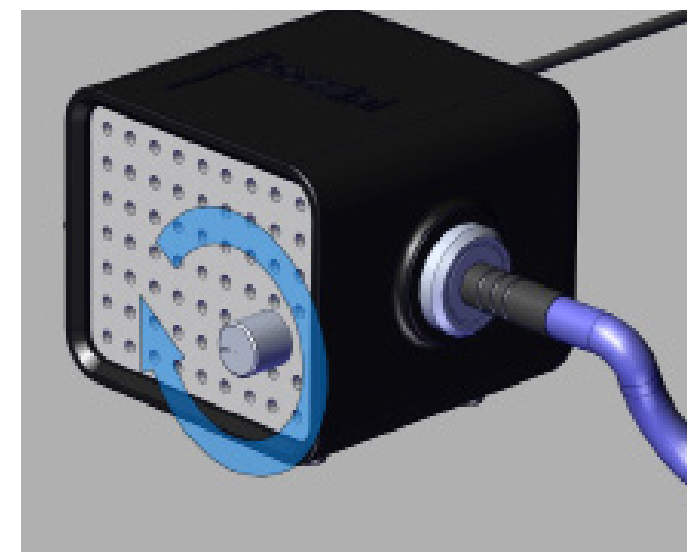


24 | When the screen is powered on, you should be able to turn the knob on the front of the light source to activate the light. Test the light at this stage and switch off.




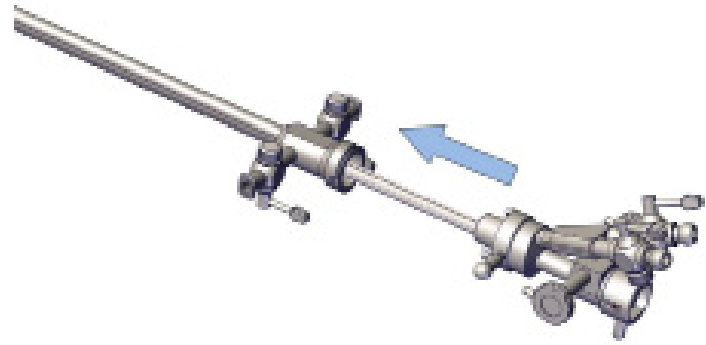


25 | Remove the introducer from the packaging. Ensure the introducer is placed inside of the sheath as illustrated here. In most cases, this step will already have been completed for you.

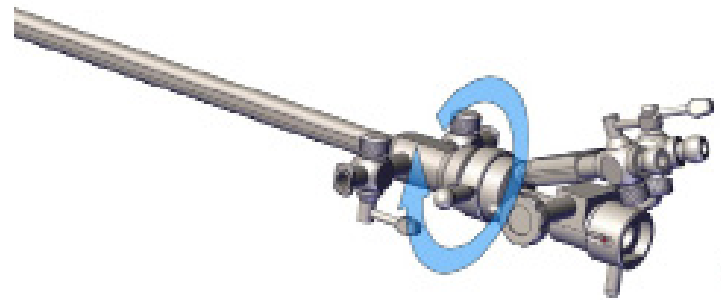


26 | Rotate the latch in a clockwise direction to ensure the two components are securely connected.

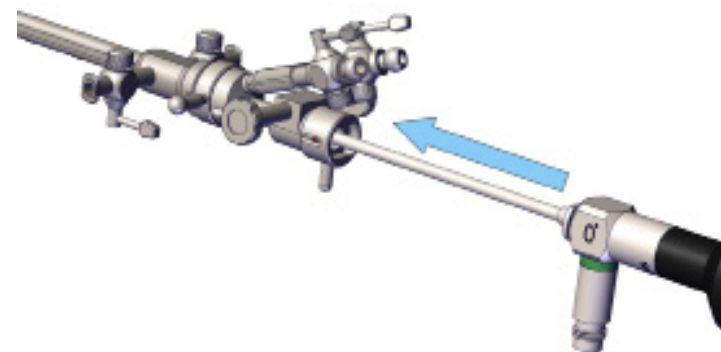


27 | You are now ready to insert the scope into the introducer. Once the scope is fully inside, ensure it is locked in place using the latch.

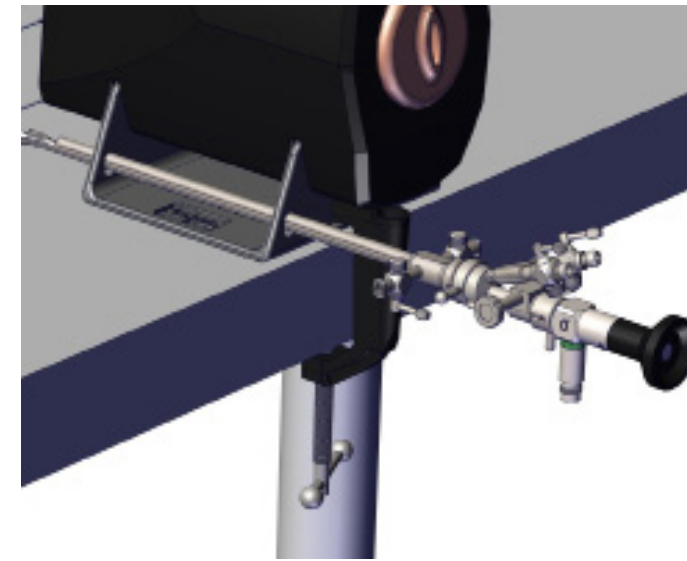

28 | You may use the apertures in the U-shaped bracket to hold the assembly in place as shown here.

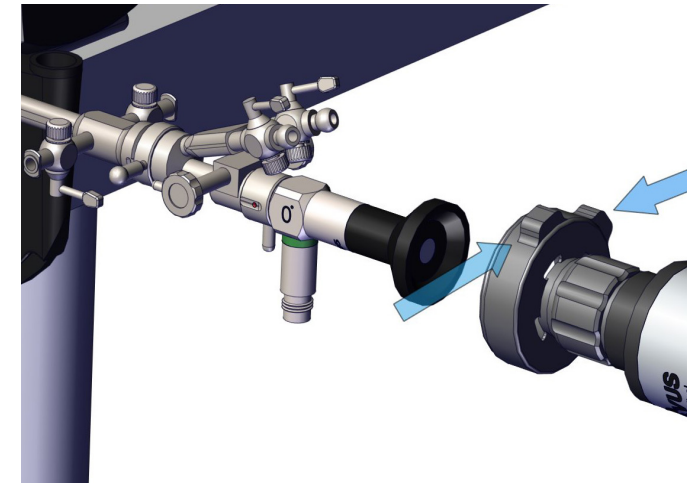

29 | Connect the endoscopic camera to the scope by pinching the two clasps as shown here.

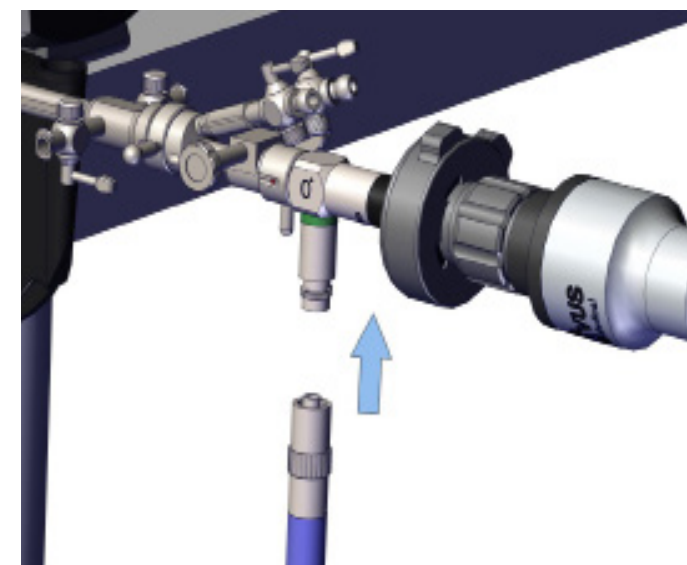

30 | Attach the other end of the fiberoptic cable to the connector on the scope by screwing it into place. At this stage, you are ready to turn the light source on.



## Surgical Training Remastered

---

Global Headquarters  
St Helens, United Kingdom  
E: [info@inovus.org](mailto:info@inovus.org)  
T: +44 (0)1744 752 952  
[www.inovus.org](http://www.inovus.org)

USA Headquarters  
Tampa, Florida USA  
E: [info@inovus.org](mailto:info@inovus.org)  
T: 512 481 7691  
[www.inovus.org](http://www.inovus.org)

   @inovusmedical

All products contained in this catalogue are Copyright Inovus Ltd 2024  
Full price list available on request: [info@inovus.org](mailto:info@inovus.org)