

LapAR™ Take Home Simulator

Specification requirements for Laptop or PC to run the LapAR™ take home software.

	Minimum specification	Recommended specification
Processor	Intel I5 CPU	Intel I7 8th Generation CPU
Graphics Card	Nvidia 1050 GPU	Nvidia 1650 GPU
Memory	6GB RAM	16GB RAM
Storage	128GB HDD/SSD	128GB HDD OR SSD

a c c e s s i b l e a f f o r d a b l e f u n c t i o n a l

E info@inovus.org

T +44 (0)1744 752 952

W www.inovus.org


   @inovusmedical

Specification check for Windows Laptop or PC

How to check Laptop spec in 4 simple steps for Compatibility with LapAR

Processor & RAM

- 1) Open Settings.
- 2) Click on System.
- 3) Click on About.
- 4) Under the Device specifications section, check the processor, system memory (RAM), architecture (32-bit or 64-bit)

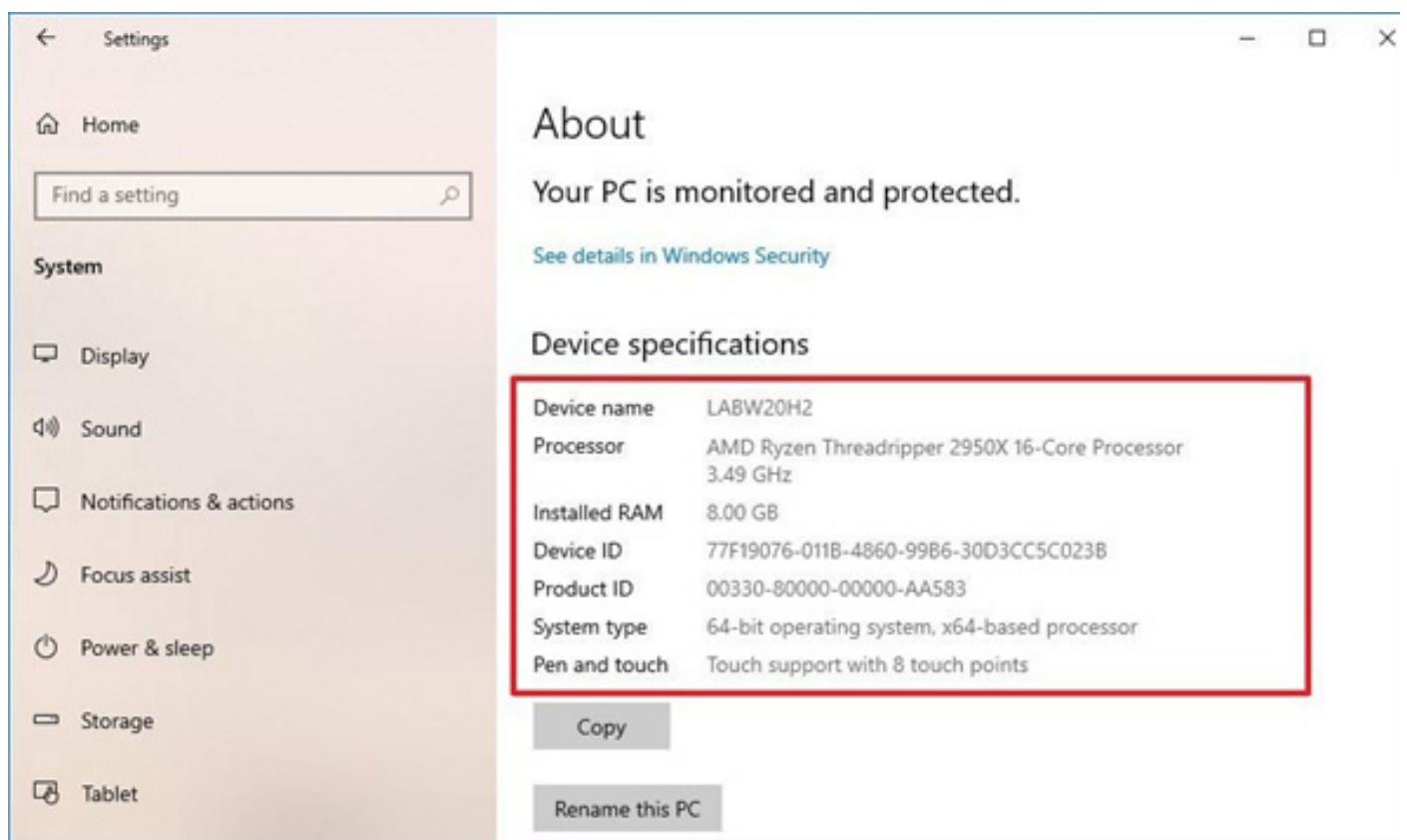

Graphics Card

- 1) Open Start.
- 2) Search for System Information and click the top result to open the tool.
- 3) Expand the Components branch on the left.
- 4) Click on Display.
- 5) Under the "Adapter Description" field, determine the graphics card in-stalled on your device.

a c c e s s i b l e a f f o r d a b l e f u n c t i o n a l


E info@inovus.org

T +44 (0)1744 752 952

W www.inovus.org

Storage

1. Select the Start button, and then select Settings .
2. Select System > Storage.

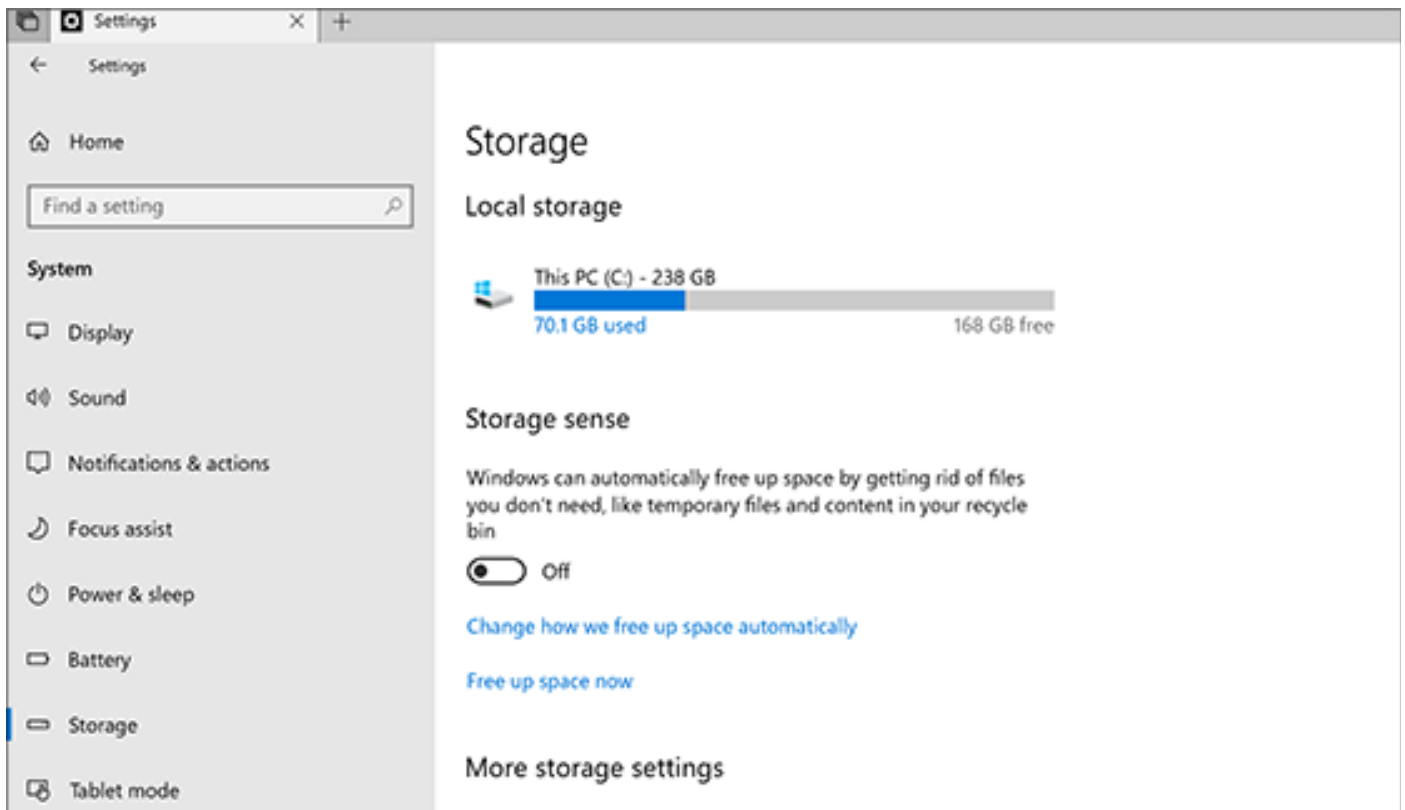

a c c e s s i b l e a f f o r d a b l e f u n c t i o n a l

E info@inovus.org

T +44 (0)1744 752 952

W www.inovus.org

